**Sterile solution for eye wash  
in single-dose container**  
Children and adults  
Preservative free  
Non-injectable

**EN Instruction for use**

**Composition**

Sodium chloride, boric acid, sodium borate, rose flavoured water, purified water qsp 5 ml.

**Product description**

Xailin® Wash comes in the form of a sterile eye wash solution in 5 ml single-dose container.

**Indications**

Xailin® Wash is indicated for:

- The treatment of surface irritations of the eyelids and the eyeball: the eyewash facilitates the removal of foreign bodies that irritate the eye, such as sand, gnats, dust, pollen, makeup debris and helps relieve the feeling of discomfort;
- Adjuvant treatment to infectious, allergic or inflammatory pathology of the eyelids, conjunctiva and cornea;
- Daily and post-operative ocular hygiene (child and adult);
- Xailin® Wash contributes to eye hydration.

**Precautions and warnings**

- Do not inject, do not swallow.
- Keep out of the reach of children.
- If you wear contact lenses you can use this product, however, you must remove them before proceeding with eye-washing.
- Single use: the single-dose container should be used immediately after opening and discarded after use.
- A single-dose container can be used for the treatment of both eyes.
- Wash hands carefully before proceeding with eye-washing.
- Avoid touching the eye or eyelids with the tip of the vial.
- The effectiveness of eye drops or an eye ointment can be disrupted by the simultaneous instillation of the eye wash solution. In case of concomitant treatment, please wait 10 minutes between each ocular instillation and always start by washing with Xailin® Wash solution unless otherwise stated by your doctor.
- If an irritation or excessive tearing or eye disorders occur or continue, stop using the solution and promptly see a doctor.

**Preservation**

Do not use after the date shown on the packaging.

Discard the single-dose container immediately after use.

Do not reuse a single-dose container that is already open.

Since this medical device contains no preservative, reusing it would involve the risk of administering a contaminated solution.

Before opening, store the single-dose containers at room temperature.

**How to use Xailin® Wash?**

2 to 8 eye washes daily.

First verify that the single-dose container has not already been used or damaged.

Then detach a single dose from its string and twist off the upper tip to open.

The washing is done directly, spouting the solution out of the container by slightly pressing onto it. It is recommended to pull the lower eyelid down, looking up, then the upper eyelid upward, looking down, in order to spread the solution over the entire eye surface to eliminate foreign bodies that irritate the eye, increase the volume of the tear film and hydrate tissues.

Xailin® Wash is a medical device of class IS.

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65NOT5513/C

AW-00125 Rev. 1

PRODUCT NAME: INST XAILIN WASH 5ML	LANGUAGES: DE EN FR IT NL RO	SIZE: 180x280 mm
MATERIAL CODE: AW-00125 Rev. 1	REVISION LEVEL: 4	ARTWORKER: J.W. Bogaards
COLOUR #1: <input checked="" type="checkbox"/> Process Black	COLOUR #4: <input type="checkbox"/>	PROGRAMME: ADOBE CC
COLOUR #2: <input type="checkbox"/>	COLOUR #5: <input type="checkbox"/>	FONTS: Myriad Pro SemiCondensed
COLOUR #3: <input type="checkbox"/>	COLOUR #6: <input type="checkbox"/>	
<b>FINAL APPROVALS</b>		
COUNTRY MANAGER #1:	COUNTRY MANAGER #2:	REGULATORY:
COUNTRY MANAGER #3:	COUNTRY MANAGER #4:	
NOTES: Several small changes upon Manufacturer's request; changed color PMS Black to Process Black		

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Xailin® is a trademark of VISUfarma