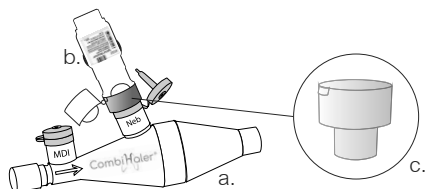
CombiHaler®

CombiHaler® mit Respimat® Adapter

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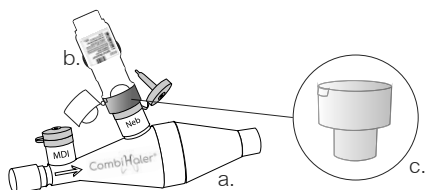
CombiHaler®

CombiHaler® with Respimat® connector

The Respimat® connector is exclusively designed to adapt to CombiHaler® spacer.

Mounting and use of Respimat® inhaler on CombiHaler® with Respimat® connector:

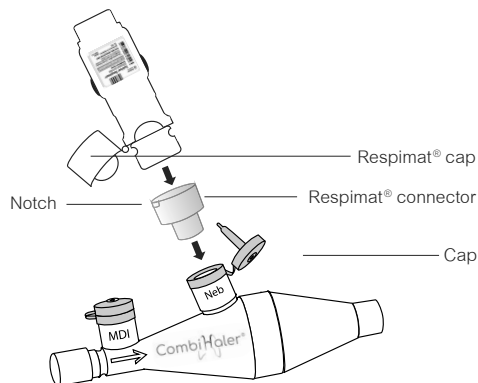
Before mounting the device, please make sure you have in your possession:

- CombiHaler® spacer
- Respimat® inhaler
- Respimat® connector

To use CombiHaler® with Respimat® connector please also follow the instructions of CombiHaler®. For the operationalization of the Respimat® inhaler, refer to Respimat® operating instructions.

CEGLA
MEDIZINTECHNIK

ASSEMBLY WITH COMBIHALER®

Open the cap of the annotated “Neb” and insert the Respimat® connector. Once the connector is correctly positioned, insert the Respimat® inhaler into the connector minding the orientation. A small notch on the side of the connector allows for the correct insertion of the inhaler. The Respimat® cap must be placed on the side of the notch.

CombiHaler® with Respimat® connector

Article-no. 381

Further information: info@ceгла.de

The trademark Respimat® belongs to Boehringer Ingelheim International GmbH.
The trademark CombiHaler® / CombiHaler Solo® belongs to Laboratoire ProtecSom SAS.
The patents pending belong to Tours University, INSERM and Laboratoire ProtecSom together.

06/2018

USE OF COMBIHALER® WITH RESPIMAT® INHALATOR

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DESCRIPTION

CombiHaler® / CombiHaler Solo® is a spacer designed for invasive and noninvasive mechanical ventilation systems. CombiHaler® allows the simultaneous use of a pressurized metered dose inhaler and a vibrating mesh nebulizer.

SPECIAL PRECAUTIONS

- CombiHaler® / CombiHaler Solo® ships fully assembled. It should never be opened before use. Any CombiHaler® / CombiHaler Solo® which has been opened should be discarded and replaced.
- When commissioning, it is necessary to ensure that:
 - The device does not contain any foreign body.
 - The caps are positioned correctly (in closed position) when no MDI and / or nebulizer is used.
 - The MDI nozzle functions properly (tested visually with a puff of product.)
- During use it is necessary to ensure that:
 - The caps are positioned correctly when no MDI and / or nebulizer is used.
 - The MDI nozzle functions properly (visual inspection during each puff of product release).

DURATION OF USE

- CombiHaler® / CombiHaler Solo® is a non-sterile device for single patient use.
- CombiHaler® / CombiHaler Solo® can be used throughout the life of the ventilating circuit for no more than seven consecutive days.
- CombiHaler® / CombiHaler Solo® should never be used on more than one circuit and must be destroyed under the same conditions as the circuit on which it was mounted.
- Any total or partial replacement of the circuit should result in the replacement of CombiHaler® / CombiHaler Solo®.

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Artikel-Nr. 381

CombiHaler® for children and adults with an opening for nebulizer

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Artikel-Nr. 382

CombiHaler®-solo for children and adults without an opening for nebulizer

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Please read through the usage directions carefully before the first use.

Die CombiHaler® / CombiHaler Solo® Verpackung beinhaltet | This CombiHaler® / CombiHaler Solo® package contains:

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The trademark CombiHaler® / CombiHaler Solo® belongs to Laboratoire ProtecSom SAS.
The patents pending belong to Tours University, INSERM and Laboratoire ProtecSom together.

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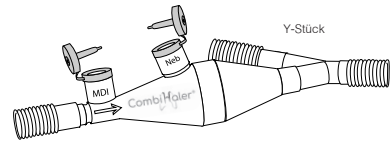
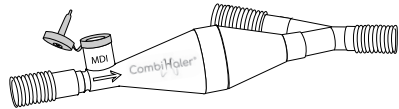
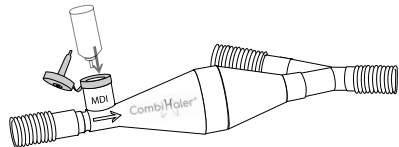
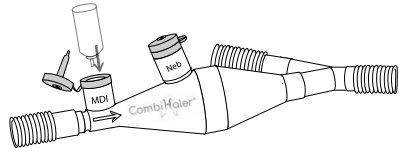
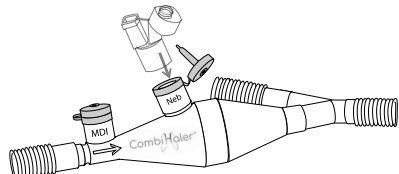
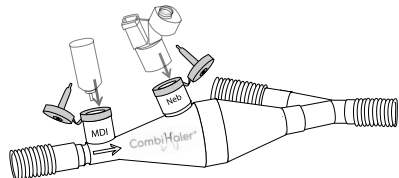
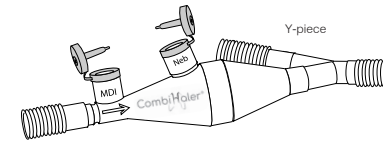
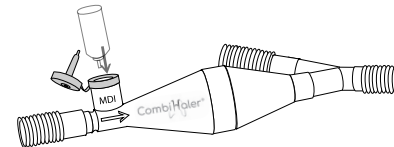
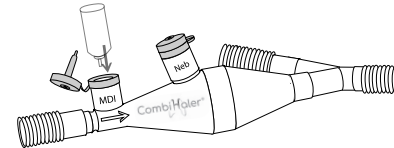
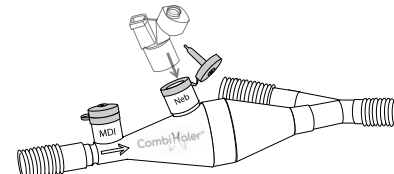
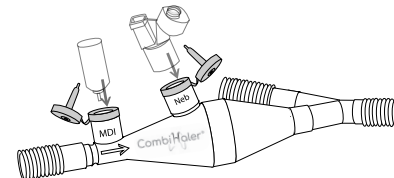
DESCRIPTION

I Mounting the circuit and use:

Insert the CombiHaler® / CombiHaler Solo® into the circuit of mechanical ventilation between the inspiratory limb and the Y-piece, following the direction of the arrow as indicated on the device CombiHaler®/ CombiHaler Solo®.

The MDI canister attaches to the port annotated MDI. For proper operation of the MDI, refer to the manufacturer indications.

Combi Haler®: The end of the nebulizer screen attaches to the orifice annotated Neb. For proper operation of the mesh nebulizer, refer to the manufacturer indications.

II Using an MDI with CombiHaler® / CombiHaler Solo®

- Shake the MDI canister according to the manufacturer's instructions.
- Open the sealing cap of the MDI port and insert the canister, valve stem facing down.
- Actuate the MDI at the beginning of the respiratory cycle.
- Let the inspiration from the ventilator administer the drug dose to the patient. Remove the canister of the MDI and replace the cap orifice in place.
- Repeat step 3 under this section II according to the number of doses needed.

III Using a vibrating mesh nebulizer with CombiHaler®

- Insert the nebulizer into the Neb. port.
- Place the drug solution in the reservoir of the nebulizer.

IV For proper operation of the nebulizer, follow the instructions provided by the manufacturer.

V CombiHaler® can be used simultaneously with both an MDI and a vibrating mesh nebulizer.